Intelli-Flo PAD FITTING INSTRUCTIONS

for Patients and Medical Professionals

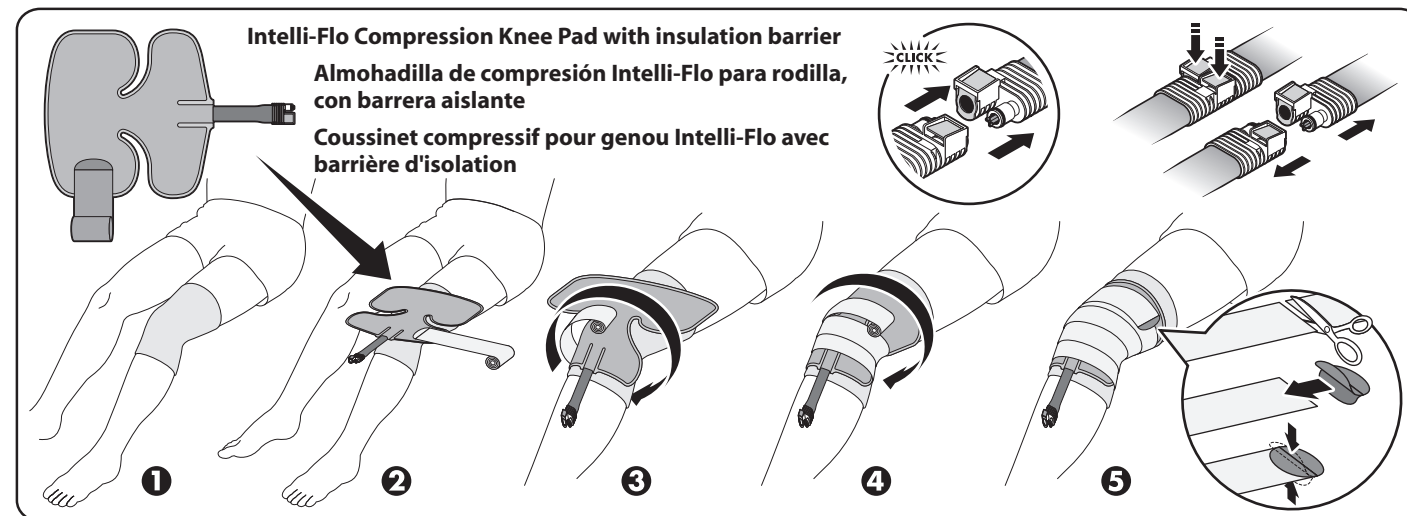
⚠ WARNING

The Intelli-Flo Pad can be cold enough to seriously injure the skin.

The Intelli-Flo Pad is non-sterile and for single patient use only. Do not attempt to sterilize the Pad. Follow these instructions and the Operating Instructions and Product Inserts provided with the Polar Care continuous-flow cold therapy units (available at www.breg.com or 1-800-321-0607).

- Discuss treatment with your licensed health care practitioner.** Provide a complete medical history including any reactions to cold. Certain medical conditions make cold-induced injury more likely. Ask your practitioner about potential adverse reactions and cold-induced injuries.
- Use only according to your practitioner's instructions** regarding the frequency and duration of cold application and breaks, proper temperature, how and when to inspect the skin, and total length of treatment. Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- Apply an insulation barrier** (such as BREG Polar Dressing, Webril, Kerlix, cast padding, elastic bandage) that provides complete coverage between the Intelli-Flo Pad and skin. **Do not let any part of the Pad touch skin.** If a sterile dressing has been applied to the treatment site that does not completely cover the skin under the Pad, use an additional insulation barrier. The Pad alone is too cold to be applied directly to the skin.
- Apply the Intelli-Flo Pad** with the plastic channeled side towards the skin. Refer to the graphics below and on the reverse side.
 - Do not wrap Pad so tightly as to restrict normal circulation, as this can cause injury.
 - Use only on a body part for which the pad is designed. Multi-Use Intelli-Flo Pads may be used on shoulders, knees, back, and hips. If the Intelli-Flo Pad is placed on a body part that it is not designed for, or if another kind of pad is used, the skin can get too cold and be injured.
- Connect the Intelli-Flo Pad** couplings to the Polar Care Kodiak unit couplings. The Intelli-Flo Pad hose couplings are only compatible with the Polar Care Kodiak unit couplings.
- Follow the Polar Care Kodiak Operating Instructions and Product Insert** inside the pouch; both are affixed to the unit.
- Inspect the skin** under the Intelli-Flo Pad (by lifting the edge) as prescribed, typically every 1 to 2 hours. Do not use the Polar Care if dressing, wrapping, bracing, or casting over the Intelli-Flo Pad prevents skin checks. Stop using and contact your practitioner immediately if you experience any adverse reactions, such as: increased pain, burning, increased swelling, itching, blisters, increased redness, discoloration, welts, other changes in skin appearance, or any other reaction identified by your practitioner.
- To end cold therapy**, disconnect electrical power and press the metal tab on each side of the hose coupling and gently pull apart or remove the Intelli-Flo Pad from your body.

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BREG Inc.
Vista, CA 92081 U.S.A.